

C-TRACT 6-Month Sample Collection Procedures:

C-TRACT

CHRONIC VENOUS THROMBOSIS
RELIEF WITH ADJUNCTIVE CATHETER-DIRECTED THERAPY



- Confirm patient consent
- Draw (2) 4.5ml blue citrate tubes of blood for research.
- Forward specimens to central lab or designated study team member for processing (within 1 hour of collection)

For questions or concerns, please contact:


- Angela Oliver, RN BSN MS, C-TRACT Trial Manager
Phone: (314) 747-8951 or Email: olivera@wustl.edu
- Sabrina Barber, Banking Core Coordinator
Phone: 314-362-9998 or Email: s.barber@wustl.edu

REMINDER:

Ensure the Collection Worksheet is completed correctly and the original form sent to the lab with the specimen.

Processing:

(2) 4.5ml citrate tubes

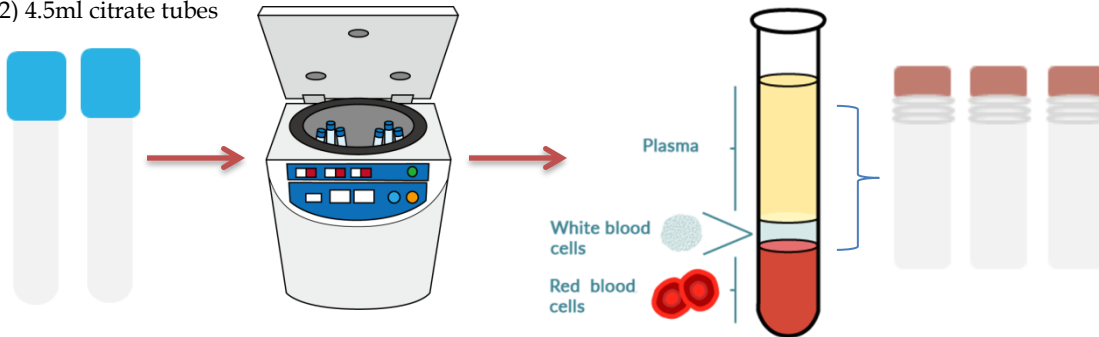

Centrifuge all tubes at **1300g*** for **5 minutes** at room temperature to separate plasma from cellular components.

Aliquot 1mL of the plasma from both citrate tubes into (3) 2mL cryovials.


REMINDER:

Ensure the vial is labeled with:

1. Date of Collection
2. Patient ID (***Do not use a hyphen or dash***)
3. Type of Sample: Buffy (B) or Plasma (P)



FedEx Overnight (Frozen) to WUSM TPC

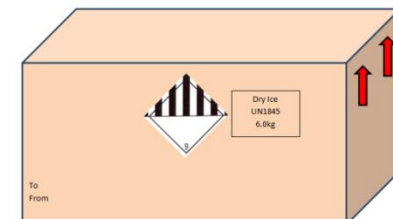

*To convert "RPM" to "gforce", consult the conversation table at: <https://www.eppendorf.com/CA-en/centrifuge-speed-calculator/>. Information from the centrifuge user manual may be required.